

CAROMA

FOR CARE 705 SUPPORT BACKREST

INSTALLATION INSTRUCTIONS


- Important Information**
- Installation must be carried out by a licensed tradesperson (plumber/builder)
 - Special care should be taken when drilling walls to avoid hidden pipes or electrical cables
 - Safety gear like a mask, earmuffs and glasses must be worn when using tools
 - It is required that these accessories are mounted to a secure and strong surface, and that the installer use appropriate fixing materials for your wall and installation. To the fullest extent permitted by law, Caroma will not accept responsibility for personal injury or damage caused by incorrect installation or poor wall strength, or installation or part installation by any person other than a licensed tradesperson (plumber/builder)
 - Clean the surface of the pad with disinfectant cleaner after use
 - Never use harsh chemicals that can damage the pad

POINTS TO CONSIDER


Cavity walls-Please ensure the backrest fixing points are secured to the wall’s solid framing structure (i.e. studwork/wall supports) to ensure adequate mounting strength of the frame.

Solid walls- For solid wall installation (i.e. brick or concrete), appropriate drilling equipment must be used.

Inside the box:

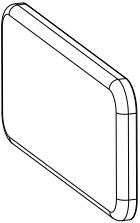


Backrest pad x1

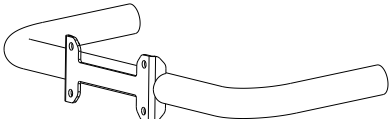

Arm x1



Cover plate x 2

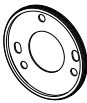

Flexible wall seal x 2



Metal base plate x 2



Ø32mm seal x 2



10G x 50 self tapping screw x 6



M8 CSK screw x 2



Backrest pad cap nuts x 4




M6 x 50 button head socket cap screw x 6



M6 x 40 shield anchor x 6



23 x 66mm hex allen key x 1

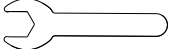


Spanner x 1

1

Backrest support frame sub-assembly

Ensure arrows are pointing up
THIS WAY UP label

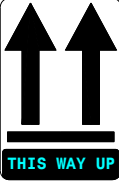


Backrest pad

M8 cap nuts

Hygiene seal assembly

Fig.1


- Tighten M8 cap nuts

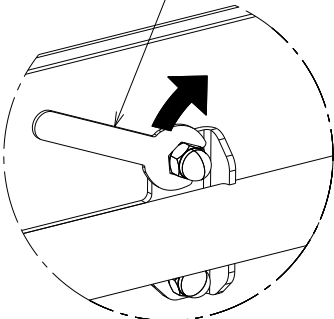

Spanner

Fig.2

2

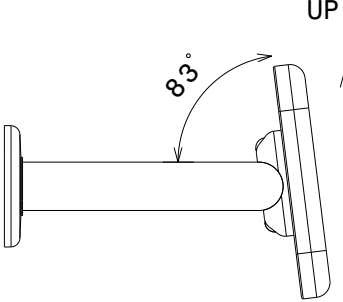


UP

83°

Fig. 3

3

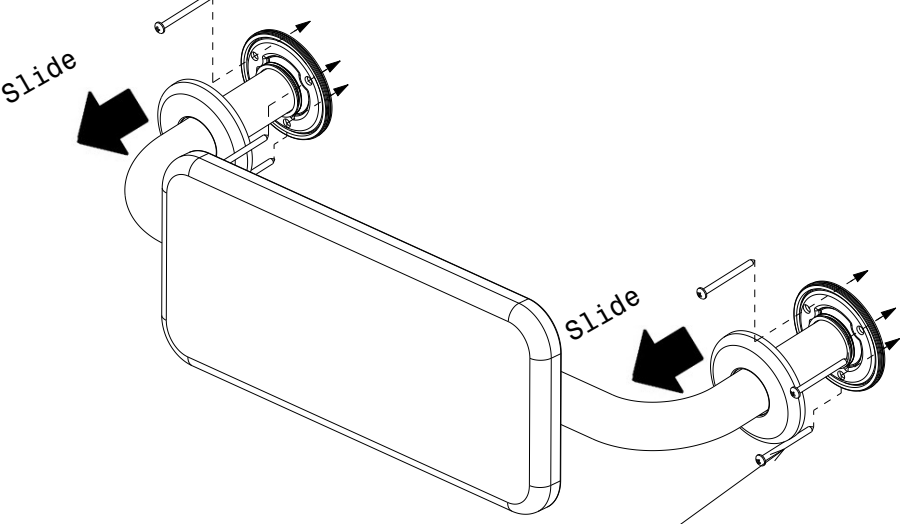


Slide

Slide

Self tapping screw

- Carefully slide the cover plates away from the seals to access the mounting plates.

On both sides, two mounting holes stay on top side.

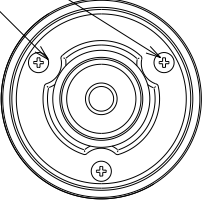


TOP

Ensure the holes on the fixing plate are orientated as shown above then mark the holes on to the wall.

Fig.4

CAROMA

FOR CARE 705 SUPPORT BACKREST
INSTALLATION INSTRUCTIONS (contd...)

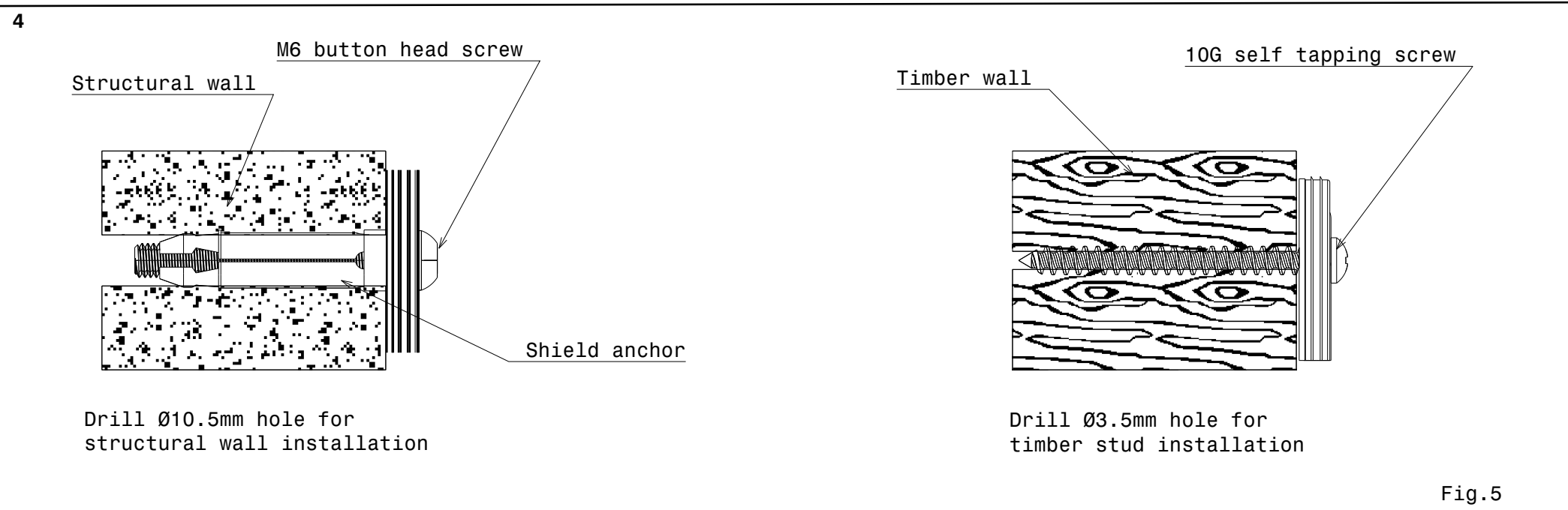

5

Installation for Invisi II Cistern with Large Access Panel

Care 705 Support Backrest

Spirit level

510

Caravelle Care DF T/Seat

- Centralise the backrest pad & frame assembly and position on the wall within the recommended height range shown in Fig. 6 (for dual flap seat) and Fig. 7 (for single flap seat).
- Check the level of frame assembly with a spirit level as shown in Fig. 6 and ensure its levelled, then mark the fixing holes on the finished wall.
- Remove the backrest pad assembly from the wall.
- Ensure the correct fasteners for the mounting and drill hole sizes are selected for timber or masonry installation as specified in Fig 5. Drill the holes at marked locations and fit the backrest assembly on the wall.
- Slide the cover plates carefully back towards the wall and push against the seal until secure.

Important:
For timber stud and non-structural walls, noggins or other suitable means of support must be used to support the backrest at the recommended height.

