
## CAROMA


## **CARE SHOWER SEAT**


## **INSTALLATION INSTRUCTIONS**

## Important Information:


- Refer to AS1428.1 Design for Access and Mobility for installation requirements
- Grade 304 Stainless Steel M8 x 70mm hexagon coach screws and 10mm wall plugs are supplied.
- For fixing to timber, use only hexagon coach screws. For fixing to brickwork, block work or concrete, use the hexagon screws with 10mm wall plugs or equivalent.

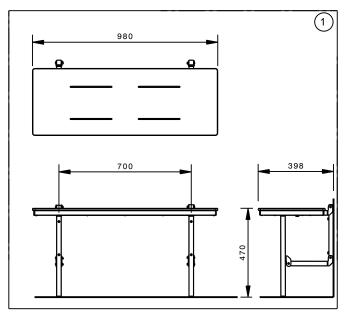

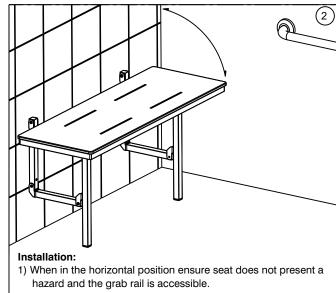


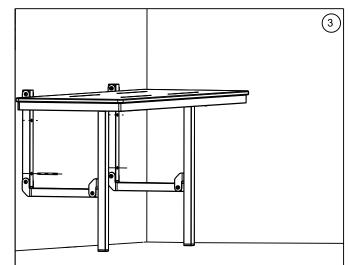

2) Unfold the seat, then while rear of the frame in contact with the wall and feet in contact with the floor, check the seat is horizontal and trace the 4 fixing holes on the wall.

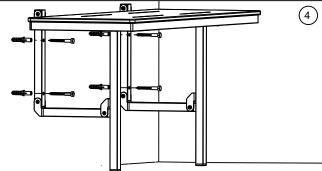

- 3) Drill Ø10mm for solid walls: (brick, masonry blocks, concrete etc.)
  Drill Ø6mm for timber stud /noggin
- a) Drill holes 50mm deep for each traced hole.
- b) (For solid walls only) Insert the wall plug into each drilled hole and ensure they are flush with the wall surface.
- Align the frame with the holes and screw M8 hexagon coach screws through until the seat frame is fully secured against the wall

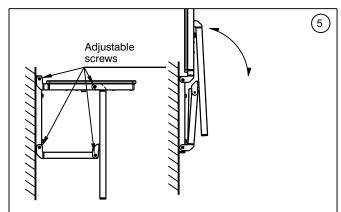

4) Check that the seat is firmly secured to wall and the tension grip hinge screws are appropriately tightened. **Important**: Check the folding of the seat a few times to ensure the seat does not drop down and can be lifted. If required the screws can be adjusted to increase or decrease the force required to open / close the seat.